

DISTEL

— c —

DOSING SYSTEM

cacheTM

TRISTEL'S CLEANING AND DISINFECTION
BRAND FOR HOSPITAL SURFACES

The DISTEL DOSING SYSTEM is intended for the disinfection of hard, non-porous medical surfaces within the hospital environment.

The DISTEL DOSING SYSTEM is effective against bacteria, yeast and viruses.

INSTRUCTIONS FOR USE

- For professional use only.
- Wear appropriate Personal Protective Equipment (PPE).
- Remove any solid waste from the surface prior to use.
- Do not use on electrical components.
- Dispose of in accordance with local policy and national regulations.

CONCENTRATE SOLUTION

Warning

Causes serious eye irritation.

Precautionary Statements

Wear protective gloves/protective clothing/eye protection.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention.



Manufactured by: Tristel Solutions Ltd
For Tristel patent information please visit
www.our-patents.info/tristel

Contact Tristel, your local distributor or visit
www.tristel.com for supporting documents such as safety data sheets, microbiological test data and reports.

1 LITRE DISTEL DOSING SYSTEM



Fill with 100ml of concentrate solution up to the first volume marker on the dosing bottle.



Fill with 900ml of ambient tap water until solution reaches the top volume marker on the dosing bottle.

500ml DISTEL DOSING SYSTEM



Fill with 50ml of concentrate solution up to the first volume marker on the dosing bottle label.



Fill with 450ml of ambient tap water until solution reaches the top volume marker on the dosing bottle.

500ml & 1 LITRE DISTEL DOSING SYSTEM



Affix the dosing cap to the dosing bottle.



Invert the bottle to fill the dosing cap.



Pour solution directly onto a surface, or onto a wipe of your choice, enough to fully saturate.



Wipe the surface. Observe a 3-minute contact time as a minimum.

cache™



Tristel Solutions Ltd, Lynx Business Park,
Fordham Road, Snailwell, CB8 7NY, UK
T +44 (0)1638 721500
E mail@tristel.com - W www.tristel.com



Tristel NV, Smallandlaan 14 B,
2660 Antwerp, Belgium

Brought to you by Tristel.



CE
2797

